

CAUTION
Before use, thoroughly read MAXAIR® CAPR® P/N 03521015 User Instruction (received with all CAPR Helmets and available at www.maxair-systems.com) for details.



REF 2071-07 **P/N** 03531148

Symbol Definitions

 Warning, Caution, or Note

REF Catalog Number **P/N** Part Number

WARNING

Failure to follow User Instructions P/N 03521015 and the instructions contained herein may be hazardous to the user's health.

Use only if package is received unopened and contents are undamaged. If damage is noted, contact the shipper for replacement or repair.

Prior to using any MAXAIR® System or component, be sure to be familiar with the system's NIOSH approved configuration.

DO NOT use if any component is damaged. If any components are damaged or contaminated and therefore unfit for safe and effective use, they should be replaced immediately.

Only trained and experienced personnel who have read and understand the User Instructions should use MAXAIR Products.

The institution using this product in any application is responsible for determining the appropriateness of this equipment relative to regulatory requirements. Bio-Medical Devices Intl, Inc. does not recommend the appropriate systems for a particular institution or facility.

Use only MAXAIR Systems/ NIOSH approved compatible components.

NOT for use in atmospheres immediately dangerous to life or health (IDLH), and atmospheres containing less than 19.5% oxygen, or more than 25% oxygen.

Flammability Level I: fabric may burn if exposed to open flame.

Follow current local regulations governing biohazard waste to safely dispose of single use MAXAIR Products.

If you need more information, contact your BMDI Sales Representative, or call BMDI customer service at 1-800-443-3842.

Intended Use

The MAXAIR 2071-07 Liner provides a soft interface, suspension, and fit adjustments for a MAXAIR Helmet to the head. The 2071-07 Liner may be used with any MAXAIR System configuration and IS THE ONLY LINER THAT MAY BE USED in a configuration that includes a Hard Hat (2061-04A).

Specifications

MAXAIR Recommended System Temperature Limits:
Use/Handling: 0°C to 54°C at a maximum 80% Relative Humidity.
Charging: 0°C to 45°C at a maximum 80% Relative Humidity.
Storage: 0°C to 35°C at a maximum 80% Relative Humidity.

Materials


Polycarbonate
Polyurethane
Polyethelene
Styrenic Block Copolymer

Regulatory

NIOSH

Liner to Helmet

 **NOTE**
The Liner attaches to the four Helmet Mounting Snaps (red arrows).

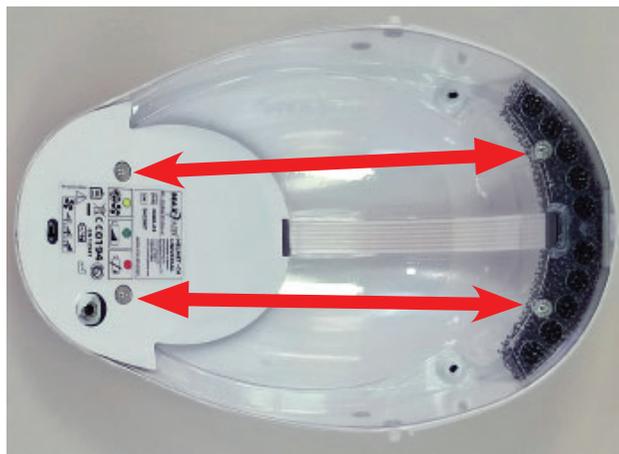

CAUTION
Before use, refer to the CAPR® User's Instructions, P/N 03521015 (received with all CAPR Helmets) for System Details.

Assembly - Liner to Helmet



1. Place Liner front (green arrow) under Helmet front (red arrow) to align Helmet Snaps with Liner Snap Holes.




2. Press Liner front right and left Snap Holes over Helmet front right and left snaps until secure.



3. Align the Helmet Power Cord with the Liner Cord cutout and press the Liner rear against the Helmet.



4. Press the Liner rear right and left Snap Holes over the Helmet rear right and left Snaps until secure.

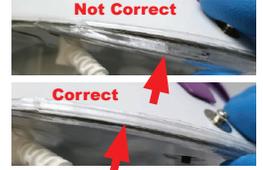

5. Ensure the Liner outer edges curls up and over the Helmet edge, all around the Helmet.

Disassembly - Liner from Helmet



1. Ensure the Power Cord is aligned with the Liner Cord cutout.



2. Grasp the front of the Liner along the top of the front Headband with the fingers.



3. With thumb against bottom front of Helmet, pull fingers down and push thumb up to disengage Front Snaps.




4. Continue pulling Liner down and away from Helmet until Rear Snaps disengage.

Liner Side Tabs

Front Button/
Long Tab


Rear Button/
Short Tab

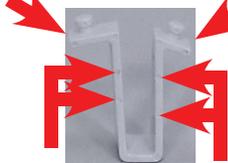


Circular hole indentations indicate this side to the outside of the Liner.

Side Tab
(one on each side of Liner.)

Front Button slot

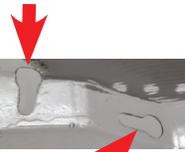


Back Button slot

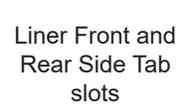


Liner Front and
Rear Side Tab
slots

Two Liner slots on each side of Liner hold front and rear Buttons of Side Tabs

Disassemble Liner Side Tabs

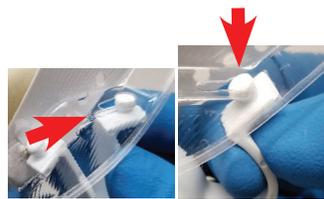

1. Push Rear Button rearward and pull down to remove from rear slot.

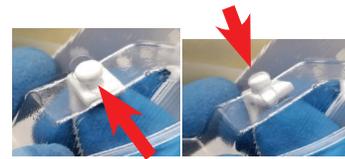

2. Push Front Button inward and pull down to remove from front slot.



3. Side Tab removed from Liner. Repeat on other side.

Assemble Liner Side Tabs



1. Insert Side Tab Front Button into Front Side Tab slot from inside, then push all the way forward (outward).



2. Insert Side Tab Rear Button into Rear Side Tab slot from underneath, then push all the way backward.



3. Properly installed Side Tab. Repeat for other side.