



### CAUTION

Before use, thoroughly read MAXAIR® CAPR® P/N 03521015 User Instruction (received with all CAPR Helmets and available at [www.maxair-systems.com](http://www.maxair-systems.com)).

P/N 2099-12




P/N 02521239 MAXAIR Screwdriver (1)	
P/N 01021596 Side Lens Mount (30)	P/N 01021595 Front Lens Mount (15)
P/N 01021407 Front Vent Clip (15)	P/N 01021408 Vent Clip Cap (15)

### Intended Use

CAPR Filter Cover Cap (FCC) Adapters are replacement adapters for the front and sides of MAXAIR 2061-08 FCCs.

### Adapter Positions

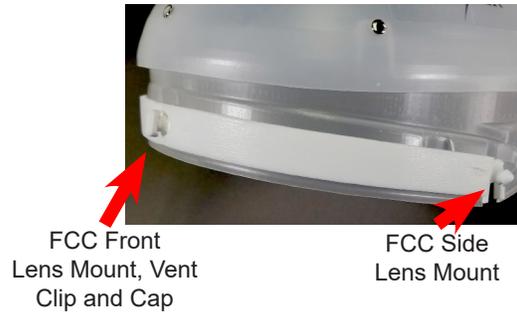

### Symbol Definitions



Warning, Caution, or Note **P/N** Part Number



### WARNING

Failure to follow User Instructions P/N 03521015 and the instructions contained herein may be hazardous to the user's health.

Use only if package is received unopened and contents are undamaged. If damage is noted, contact the shipper for replacement or repair.

Prior to using any MAXAIR® System or component, be sure to be familiar with the system's NIOSH approved configuration.

DO NOT use if any component is damaged. If any components are damaged or contaminated and therefore unfit for safe and effective use, they should be replaced immediately.

Only trained and experienced personnel who have read and understand the User Instructions should use MAXAIR Products.

The institution using this product in any application is responsible for determining the appropriateness of this equipment relative to regulatory requirements. Bio-Medical Devices Intl, Inc. does not recommend the appropriate systems for a particular institution or facility.

Use only MAXAIR Systems/ NIOSH approved compatible components.

NOT for use in atmospheres immediately dangerous to life or health (IDLH), and atmospheres containing less than 19.5% oxygen, or more than 25% oxygen.

Flammability Level I: fabric may burn if exposed to open flame.

Follow current local regulations governing biohazard waste to safely dispose of single use MAXAIR Products.

If you need more information, contact your BMDI Sales Representative, or call BMDI customer service at 1-800-443-3842.

### Filter Cover Cap (FCC) Configurations

There are two primary versions of 2061-08 FCCs that are fully interchangeable.



### Remove Current Front Lens Mount



1. From inside the FCC, pinch the legs of the Front Vent Clip together with the MAXAIR Screwdriver flat blade.



2. Push the Front Vent Clip fully out of the Front Lens Mount.



3. From outside the FCC, insert the Phillip blade through the Front Lens Mount.



4. Bracing the front of the FCC from the inside, push the screddriver inward with controlled force to push the Front Lens Mount out of the FCC.



### Attach Vent Clip Cap to Front Vent Clip



1. Place Vent Clip Cap over top of Vent Clip



2. Press Vent Clip Cap into top of Vent Clip firmly so that the top surface of the two parts are even and flat.

### Replace Front Lens Mount



1. From inside the FCC, place a new Front Lens Mount into the front hole on the FCC.



2. Position the Front Lens Mount as shown at A.



3. From outside the FCC, position the legs of a Front Vent Clip into the front opening of the Front Lens Mount.



4. Compress the legs of the Front Vent Clip by wedging them into the opening of the Front Lens Mount.



5. Press the Front Vent Clip fully into the Front Lens Mount.

### Remove Side Lens Mount



Grasp the Side Lens Mount between thumb and forefinger and pull down and away from the FCC.

### Replace Side Lens Mount



1. Grasp a Side Lens Mount between thumb and forefinger and slide it into the Side Lens Mount slot in the FCC.



2. Press the Side Lens Mount fully up into the slot as shown in A.

