



CAUTION

Before use, thoroughly read MAXAIR® CAPR® P/N 03521015 User Instruction (received with all CAPR Helmets and available at www.maxair-systems.com).


P/N 2099-14



Intended Use

MAXAIR Hard Hat Adapters are replacement adapters for the front and sides of MAXAIR 2061-04A Hard Hats.

Hard Hat Adapter Positions

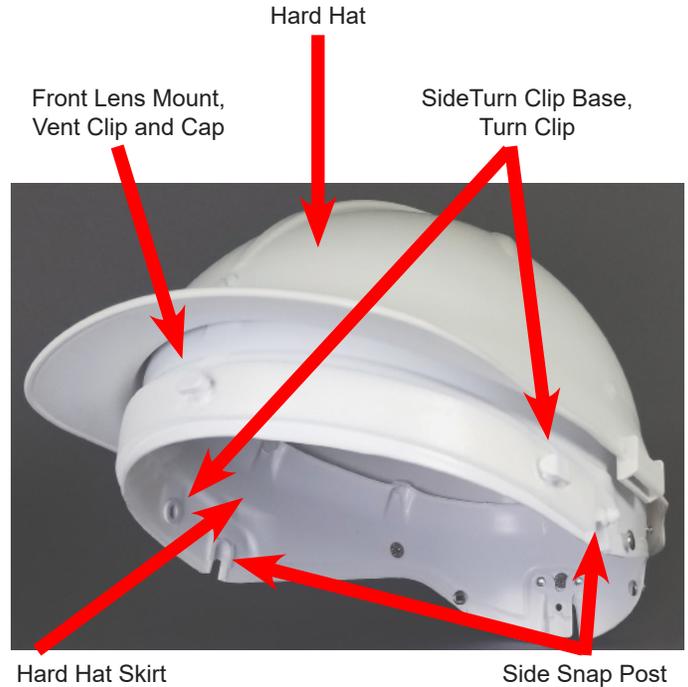

Symbol Definitions



Warning, Caution, or Note **P/N** Part Number



WARNING

Failure to follow User Instructions P/N 03521015 and the instructions contained herein may be hazardous to the user's health.

Use only if package is received unopened and contents are undamaged. If damage is noted, contact the shipper for replacement or repair.

Prior to using any MAXAIR® System or component, be sure to be familiar with the system's NIOSH approved configuration.

DO NOT use if any component is damaged. If any components are damaged or contaminated and therefore unfit for safe and effective use, they should be replaced immediately.

Only trained and experienced personnel who have read and understand the User Instructions should use MAXAIR Products.

The institution using this product in any application is responsible for determining the appropriateness of this equipment relative to regulatory requirements. Bio-Medical Devices Intl, Inc. does not recommend the appropriate systems for a particular institution or facility.

Use only MAXAIR Systems/ NIOSH approved compatible components.

NOT for use in atmospheres immediately dangerous to life or health (IDLH), and atmospheres containing less than 19.5% oxygen, or more than 25% oxygen.

Flammability Level I: fabric may burn if exposed to open flame.

Follow current local regulations governing biohazard waste to safely dispose of single use MAXAIR Products.

If you need more information, contact your BMDI Sales Representative, or call BMDI customer service at 1-800-443-3842.

2099-14 Kit Components

P/N 02521239 MAXAIR Screwdriver (1)	
P/N 01021595 Front Lens Mount (15)	P/N 01021408 Vent Clip Cap (15)
P/N 01021407 Front Vent Clip (15)	P/N 01021404 Side Turn Clip Base (30)
P/N 01021596 Side Lens Mount (30)	P/N 01021406 Turn Clip (30)

Remove a Front Lens Mount and Vent Clip From Hard Hat (HH)



1. From inside the HH, pinch the legs of the Front Vent Clip together with the MAXAIR Screwdriver flat blade.



2. Push the Front Vent Clip fully out of the Front Lens Mount.



3. From outside the HH, insert the Phillip blade through the Front Lens Mount.



4. Bracing the front of the HH from the inside, push the screddriver inward with controlled force to push the Front Lens Mount out of the HH.

Attach Vent Clip Cap to Front Vent Clip



1. Place Vent Clip Cap over top of Vent Clip




2. Press Vent Clip Cap into top of Vent Clip firmly so that the top surface of the two parts are even and flat.

Assemble a Front Lens Mount to the HH



1. From inside the HH, place a new Front Lens Mount into the front hole on the HH.

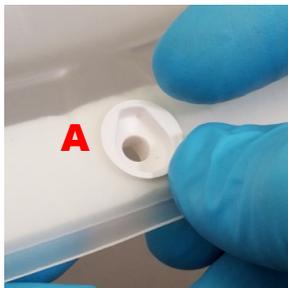

2. Position the Front Lens Mount as shown at A.



3. From outside the HH, position the legs of a Front Vent Clip into the front opening of the Front Lens Mount.



4. Compress the legs of the Front Vent Clip by wedging them into the opening of the Front Lens Mount.




5. Press the Front Vent Clip fully into the Front Lens Mount.


Remove a Side Lens Mount (Same for both sides of HH)



1. Locate a Side Lens Mount

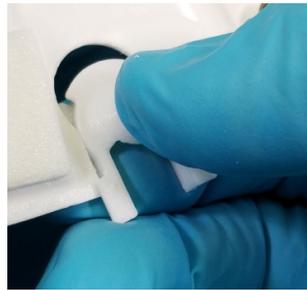

2. Grasp the Side Lens Mount between thumb and forefinger and pull down and away from the HH.

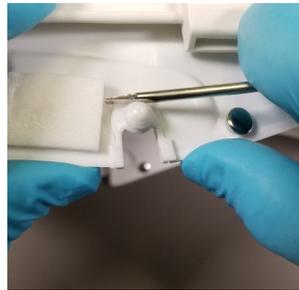


Alternative to step 2, position both thumbs at bottom edge of HH Skirt for support, and use the screwdriver flat blade on top of the protruding post and press down to remove the Lens Mount.

Assemble a Side Lens Mount (Same for both sides of HH)



1. Grasp a Side Lens Mount between thumb and forefinger and slide it into the Lens Mount slot in the HH.

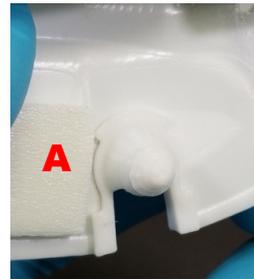

2. Press the Side Lens Mount fully up into the slot as shown in A. Alternatively, use the Screwdriver Flat Blade as leverage to push the Lens Mount fully into the Lens Mount Slot.

Remove Side Turn Clip and Base (Same for both sides of HH)



1. Locate the Side Turn Clip and Base.



2. From inside the HH, pinch the legs of the Side Turn Clip together with the MAXAIR Screwdriver flat blade, and push the turn Clip out of the HH Skirt.



3. If the Turn Clip is only partially out from step 2., it should be out enough to manually pull it fully out of the skirt from the outside.



4. From outside the HH, insert the Phillip blade through the Side Turn Clip Base.




5. Bracing the the HH from the inside, push the screwdriver inward with controlled force to push the Side Turn Clip Base out of the HH.

Assemble a Side Turn Clip and Base (Same for both sides of HH)



1. From inside the HH, place a new Side Turn Clip Base into the round side hole on the HH.

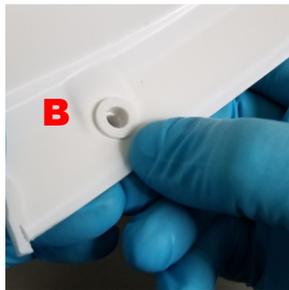

2. Press the Turn Clip Base fully into the HH Skirt as at B.



3. From outside the HH, use the screwdriver flat plat to aid in positioning the legs of a Side Turn Clip into the front opening of the Side turn Clip Base.



4. Press the Turn Clip fully into the Clip Base.